

# LIFEPAK® 12, 20 OR 20e DEFIBRILLATOR/MONITORS

---

## Why can I only view Paddles lead or Lead II in Channel 1 on my LIFEPAK 12 or LIFEPAK 20/20e defibrillator/monitors?

---

### Explanation:

When the VF/VT alarm is on, you are limited to Paddles lead or Lead II in Channel 1. If you want to view other leads, you can turn off the alarm.

On the LIFEPAK 12 monitor, if the device is in Advisory Monitoring Mode, it is equivalent to having the VF/VT alarm on. An “Advisory Mode - Monitoring” status message displays on the bottom of the screen. The 20/20e does not have this mode of operation.

### Steps to resolve:

While in Manual Mode, turn off the VF/VT Alarm.

- a. To temporarily turn off the VF/VT alarm while monitoring, press the Alarms button on the right side of the device front panel. Select VF/VT Alarm and select OFF.
- b. To permanently turn off the VF/VT alarm, go into Setup mode. Select the Alarms menu, select the VF/VT Alarm menu and select OFF.

For instructions on Setup mode, refer to Operating Instructions, Section 8 for the LIFEPAK 20/20e defibrillator/monitor and Section 9 for the LIFEPAK 12 device.

### References:

LIFEPAK 20e Defibrillator/Monitor, LIFEPAK 20 Defibrillator/Monitor and LIFEPAK 12 Defibrillator/Monitor Operating Instructions

### Applies to:

LIFEPAK 12 Defibrillator/Monitor  
LIFEPAK 20e Defibrillator/Monitor  
LIFEPAK 20 Defibrillator/Monitor

**For further information, please contact your Physio-Control representative at 800.442.1142 or visit our website at [www.physio-control.com](http://www.physio-control.com)**