


LIFEPAK CR® Plus / LIFEPAK EXPRESS® DEFIBRILLATORS

How do I replace the QUIK-PAK™ electrode packet on my LIFEPAK CR Plus or LIFEPAK EXPRESS defibrillator?

Explanation:


Both the QUIK-PAK electrode packet and the CHARGE-PAK™ battery charger should be replaced at the same time after each use of the defibrillator or when the readiness display shows the  symbol.

Steps to resolve:


1. Press the ON/OFF button to open the defibrillator lid (voice instructions will sound).



2. Press and hold down the ON/OFF button for 2 seconds to turn off the defibrillator voice prompts and save battery power.

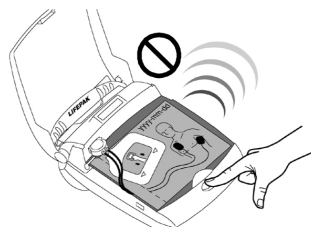


3. To remove the old electrode packet, unplug the electrode connector from the receptacle.

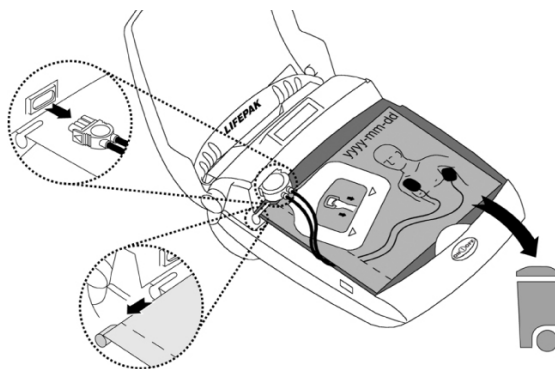


4. Slide the anchor pin from the slot.

5. Discard the outdated or used electrode packet according to local regulations or in the trash.


6. To install the new electrode, slide the anchor pin into the slot.

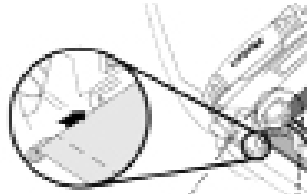


7. Plug the electrode connector into the receptacle.

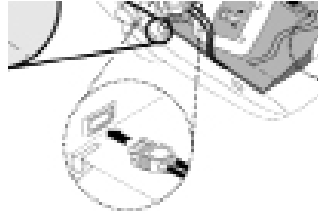

8. Ensure the new electrode packet is centered on the defibrillator and tucked behind the lip before closing the lid.

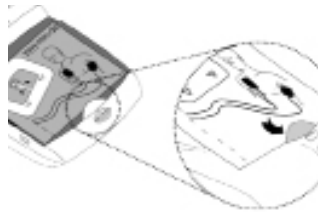

9. Close the lid. Confirm that the packet "Use By" date is visible through the window in the upper right-hand corner of the lid.

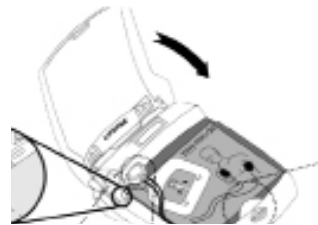

Reference:

LIFEPAK CR Plus and LIFEPAK EXPRESS Defibrillator Operating Instructions, Section 5.3

Applies to:

LIFEPAK CR Plus Defibrillator
LIFEPAK EXPRESS Defibrillator

For further information, please contact your Physio-Control representative at 800.442.1142 or visit our website at www.physio-control.com